Casa Elite Bed Extension – Instructions For Use

INTRODUCTION

Thank you for purchasing this accessory. These instructions for use cover CASA/BED/EX. This accessory is only intended for use with the Casa Elite and Casa Elite Low. These instructions and the bed instructions for use should be read carefully and fully understood before use. Please ensure the length extension has been fitted correctly and securely by authorised personnel only. If you have any questions concerning the operation or maintenance, please contact your provider/supplier who will provide you with expert professional advice.

- Before attempting to assemble the extension onto the bed ensure these instructions and the bed instructions for use have been read and fully understood.
- Ensure the bed is unoccupied prior to fitment/removal of the accessory.
- Ensure a risk assessment in line with your local health and safety policy is undertaken.
- Never attempt to fit the extension when side rails are fitted- risk of side rails falling and creating a crushing hazard. Always remove the side rails first.




- If the bed is in its extended state and with the leg extension profiled an opening can be created at the foot end of the bed; a patient risk assessment must be performed to assess if this poses a potential hazard to the patient – if so the leg section must be locked out prior to use.
- Drive DeVilbiss Healthcare recommend that the extension is only to be used for individuals who require it due to their physical height requirements, it is not to be fitted to the bed as the default for all occupants. A patient risk assessment must be performed to assess the need for the extension.

INSTALLATION

- 1) Flatten all sections and lower the bed to its minimum height (see bed instructions for use for handset operation).
- 2) Ensure the head end brakes are applied.
- 3) Unclip the foot end actuator cable from the cable clip on the bed frame to prevent overextending the cable.
- 4) At the foot end of the bed, use the Allen key supplied (for Allen key storage location see bed instructions for use) to loosen the 4 screws retaining the bed end to the leg section.
- 5) Remove the bed end from the leg section, keep nearby avoid stretching/damaging the actuator cable. Alternatively remove the cable from the control box, see bed instructions for use for cable removal. Be sure to support the leg section whilst removing the bed end. For ease, the help of a second able bodied person should be considered.
- 6) Slide the extension onto the bed end, tightening all screws to secure (Figure 1).
- Slide the bed end complete with the extension back into the leg section (A), secure into place with the screws located on the leg section (B) (Figure 2).



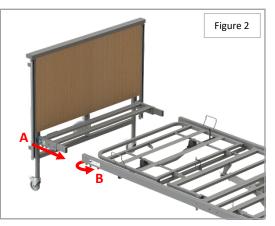

SYMBOL DEFINITION

LOT	Lot number	REF	Catalogue number
\sim	Date of manufacture		Manufacturer
	Refer to instructions for use - Mandatory Failure to read the instructions for use could introduce a hazard		

DECONTAMINATION

Infection control and routine cleaning must be carried out in accordance with your local infection control policy or regulatory body. For full decontamination guidelines please see the bed instructions for use.

Always disconnect the bed from the mains power supply prior to cleaning.
Do not allow fluids to penetrate the unit.
Regular cleaning and disinfection will help to prevent risk of infection to the occupant and/or career, prior to transferring the accessory to another user ensure it has been cleaned and disinfected according to this and the bed instructions for use.

WARRANTY

Drive DeVilbiss Healthcare Ltd. warrants that this product will perform in accordance with its specification and will remain free from defects in material and workmanship when used under normal conditions for a period of 1 year (full parts and labour) from the date of purchase from Drive DeVilbiss Healthcare Ltd. If purchased from an authorised dealer or international distributor, the product is warranted for 1 year parts only.

DRIVE DEVILBISS HEALTHCARE LTD. MAKES NO OTHER WARRANTIES, EXPRESS OR IMPLIED, AND ALL IMPLIED WARRANTIES OF MERCHANTABILITY, NON-INFRINGEMENT AND FITNESS FOR A PARTICULAR PURPOSE ARE HEREBY DISCLAIMED. IN NO EVENT WILL DRIVE DEVILBISS HEALTHCARE LTD. BE LIABLE FOR PUNITIVE, SPECIAL OR CONSEQUENTIAL DAMAGES, OR FOR AN AMOUNT IN EXCESS OF THE PURCHASE PRICE OF THE DEFECTIVE DRIVE DEVILBISS HEALTHCARE LTD. PRODUCT OR PRODUCTS.

This limited warranty of up to 1 year gives you specific legal rights and you may also have other rights.

Drive DeVilbiss Healthcare Ltd. has a policy of continual product improvement and reserves the right to amend specifications covered in this document. Please contact Drive DeVilbiss Healthcare Ltd. for complete warranty information.

No part of this document may be reproduced without the written approval of Drive DeVilbiss Healthcare Ltd.



Drive DeVilbiss Healthcare Ltd., Sidhil Business Park, Holmfield, Halifax, West Yorkshire, HX2 9TN United Kingdom Tel: +44(0) 845 0600 333 Fax: +44(0) 845 0600 334 info@drivedevilbiss.co.uk



www.drivedevilbiss.co.uk