User Guide Universal Cylinder Trolley – 1531




Introduction

Thank you for purchasing a Sidhil universal cylinder trolley. Your cylinder trolley is supplied fully assembled and ready for use. The trolley is universal, having three different settings allowing it to cater for the following oxygen cylinders:

AD, DD, PD, RD, D, E, AF, DF, F, LF, VF, AV, HX, T4, T7 & G

- 1. Hand Grip
- 2. Handle Tube
- 3. Hand Wheel
- 4. Cover Cap
- 5. Castor Wheel
- 6. Top Yoke
- 7. Straps
- 8. Base Plate

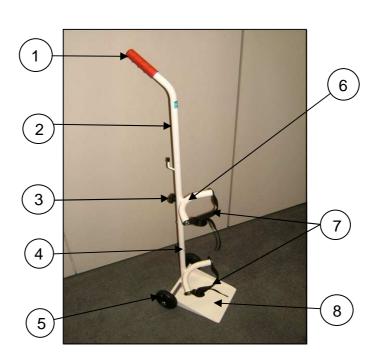

Specifications

- Net Weight (Empty): 3.6kg (8lb)
- Overall Height: 1060mm (41.7")
- Overall Width: 300mm (11.8")
- Overall Depth: 290mm (11.4")
- Min Cylinder Diameter: 90mm (3.5")
- Max Cylinder Diameter: 200mm (7.9")

Preparing for use

Before use, ensure all parts are present; see above for the parts list.

- 1. Choose the required height dependant upon the cylinder size for the top yoke.
- 2. Recommended yoke position, is as high as possible without being above the neck of the cylinder.
- 3. Position the yoke ensuring the bolt thread is inserted through the handle tube.
- 4. Screw the hand wheel onto the thread and firmly tighten.
- 5. Press the cover caps into the open holes to blank them off.

Positioning The Yoke:



Looping Strap Through Buckle:

- 6. Place the cylinder onto the base plate ensuring it is fully supported by both yokes.
- 7. Fasten in place by tightening the straps around the cylinder.
- 8. Ensure the strap is threaded through the buckle as shown.
- 9. Before use ensure the cylinder is SECURE.



Inspection

The cylinder trolley requires very little maintenance, however if there are any signs of damage, or it is not performing as it should, please contact your supplier for assistance.

- Check there are no loose/missing parts or fixings.
- Check no parts appear excessively worn.
- Check that the straps are not frayed in any way.

Cleaning & Maintenance

All surfaces to be wiped down with a disposable soft cleaning cloth, moistened with a mild detergent and diluted in hot water (60° C). Extra care should be taken around areas where excess dirt or dust may gather. Rinse down with clean water and dry off with a paper towel.

Always ensure that the cylinder trolley is dry before re-using.

Sidhil do not recommend the use of concentrated bleach or abrasive cloths, as these could mark the surfaces.

Warranty

Sidhil Limited standard terms and conditions apply to all purchases; a copy is available from the address below. Your statutory rights are not affected.



Sidhil Limited, Sidhil Business Park, Holmfield, Halifax, West Yorkshire HX2 9TN

Tel: +44 (0)1422 233030 Fax: +44 (0)1422 233040 Email: info@sidhil.com Sales Tel: +44 (0)1422 233030 Sales Fax: +44 (0)1422 233040



A Member of the Siddall & Hilton Ltd Group of Companies

INSTRUC-1531 rev 0 20/09/07